





EZ9Plus & EZ11Plus User Self Help: "Heat time error"



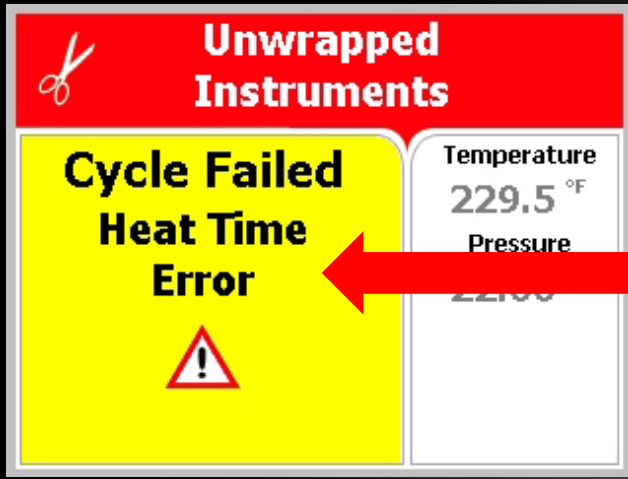

Problem:


This message is displayed if the chamber cannot reach the required temperature within the preset time.

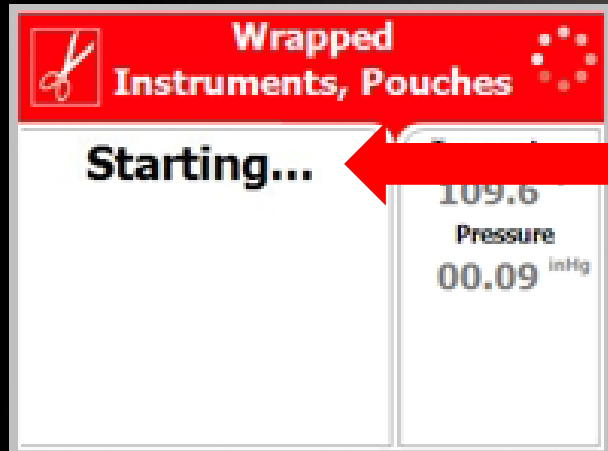

What to do:

Perform a new cycle.



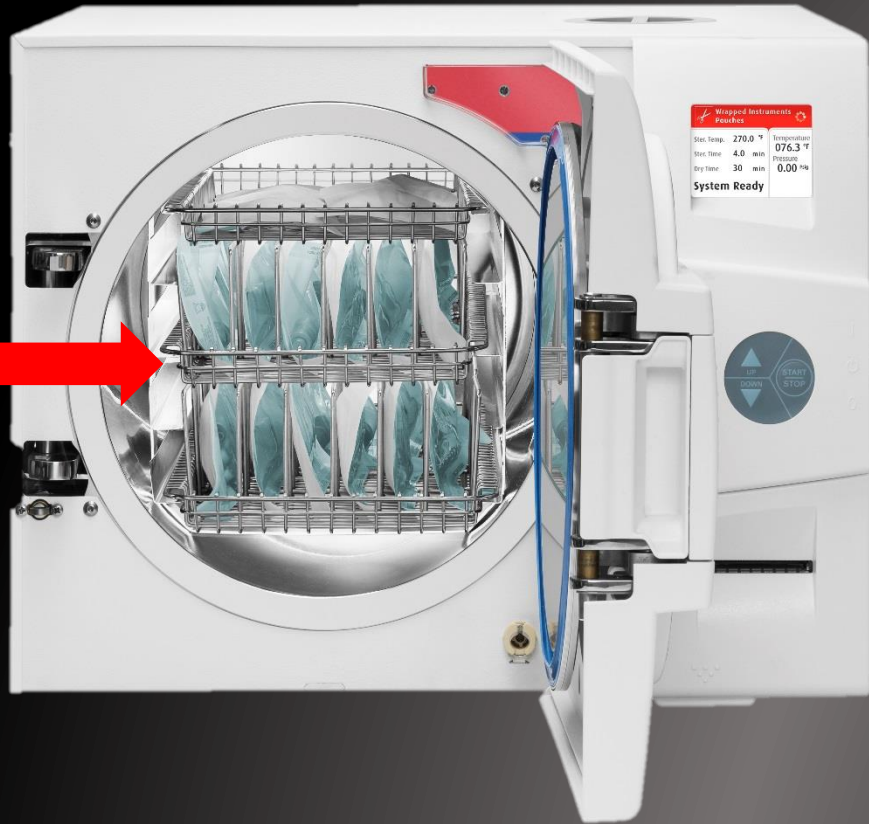

What to do:
The chamber may be overloaded, remove some material from the chamber.



**Tuttnauer**[™]
Your Sterilization & Infection Control Partners



What to do:
The cassette needs to be
vertical in the EZ11Plus





Problem:


The door may not be closed properly

What to do:

On your next cycle close the door more forcefully

Door gasket may be dirty or faulty. See video



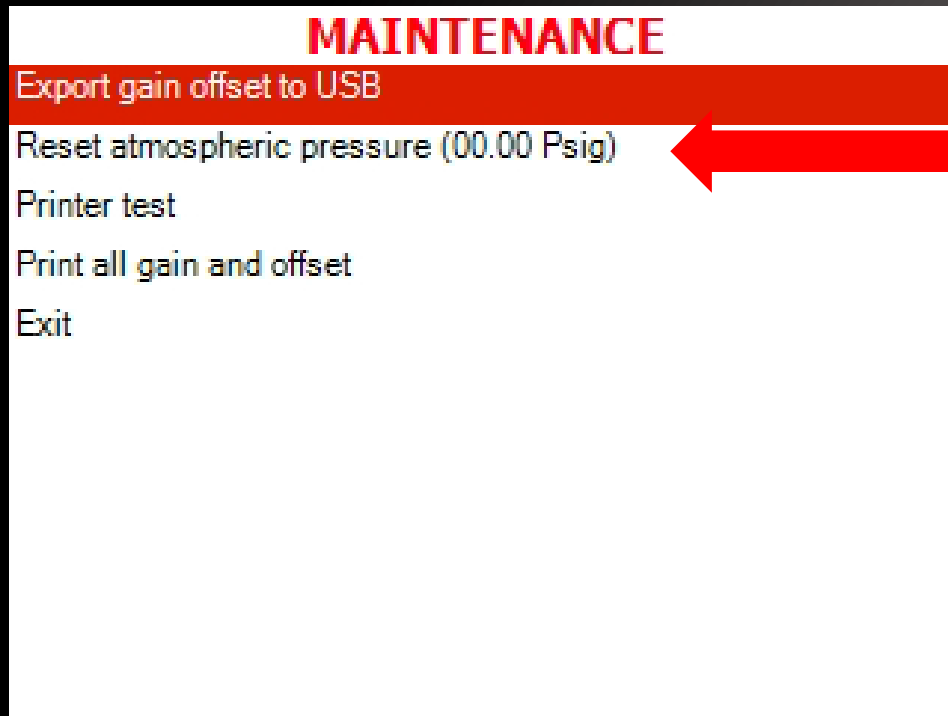

What to do:
The atmospheric pressure parameter may need to be set.
See Atmospheric Pressure Parameter reset video.

If the problem persists call your dealer or Tuttnauer USA for service.

