

T-Edge Autoclave

The only gravity autoclave
upgradeable to pre/post vacuum
by software!

- Larger chamber capacity & shorter cycle times
- User-friendly touchpad & control panel
- Advanced documentation & traceability options
- Available in 10" & 11" Chambers



Designed to meet all current
sterilization standards ANSI/AAMI ST55


Tuttnauer
Innovation · Legacy · Partnership

SINCE
1925

Features:

- Optimal chamber capacity
- Convenient and easy loading with door opening to 110°
- Assorted water filling and drainage methods (top, front or automatic)
- Range of automatic one-button programs
- Remote Wi-Fi monitoring via smartphone or tablet

Printer includes sterilization data (including expiration date).



Technical Specifications

Autoclave Model	Chamber Size (Inches)	Volume	Overall Dimensions D x W x H (Inches)	No. of Trays	Connections	Standard Cassette Capacity	Item Supplies	Voltage (V) Freq. (Hz)	Weight
T-Edge10 220V	18" depth x 10" diameter	6 gal/ 23 liters	21.1"x19" x19.7"	5	1 ethernet port, 4 USB ports, WiFi, intergrated automatic water filling and drain connection	3 Full + 3 Half	Drain tube, tray holder	230V, Frq 50-60Hz	117 lbs.
T-Edge10 110V	18" depth x 10" diameter	6 gal/ 23 liters	21.1"x19" x19.7"	5	1 ethernet port, 4 USB ports, WiFi, intergrated automatic water filling and drain connection	3 Full + 3 Half	Drain tube, tray holder	120V, Frq 50-60Hz	117 lbs.
T-Edge11 110V	18" depth x 11" diameter	7 gal/ 27 liters	21.1"x20" x19.7"	5	1 ethernet port, 4 USB ports, WiFi, intergrated automatic water filling and drain connection	4 Full + 4 Half	Drain tube, tray holder	120V, Frq 50-60Hz	124 lbs.

Sterilization Programs

Class S	Icon	Cycle Program	Load Type	*T-Edge10 Cycle Times Including Drying (Minutes)	*T-Edge11 Cycle Times Including Drying (Minutes)
		Unwrapped Instruments 270° F	Unwrapped Instruments	20 Minutes	25 Minutes
		Wrapped Pouches 270° F	Wrapped Pouches	32 Minutes	43 Minutes
		Unwrapped Delicate 250° F	Unwrapped Delicate Instruments	45 Minutes	47 Minutes
		Handpieces 270° F	Handpieces	45 Minutes	59 Minutes

Class B	Icon	Cycle Program	Load Type	*T-Edge10 Cycle Times Including Drying (Minutes)	*T-Edge11 Cycle Times Including Drying (Minutes)
		Unwrapped Instruments 273° F	Unwrapped Instruments	20 Minutes	26 Minutes
		Wrapped Pouches 273° F	Wrapped Pouches	39 Minutes	49 Minutes
		Unwrapped Delicate 250° F	Unwrapped Delicate Instruments	45 Minutes	47 Minutes
		Wrapped Delicate 250° F	Wrapped Delicate Instruments	58 Minutes	75 Minutes

*Cycle times are approximate, and may vary based on the instruments load, heating, sterilization and drying.

For Your T-Edge

Chamber Clean:
CLEAN & SHINE
Your Chamber
Tablets CB0030
For T-Edge Only



T-Clean Chamber Shine
SH0006
500 ml with sprayer



Biological Readers & Indicators

WTL198-0072
Ultra Rapid
20 Minute
Biological
Indicator

20 MIN



WTL198-0058
Rapid 1 Hour
Biological
Indicator

1 HOUR



WTL198-0057
Mini
BioNova
Reader



WTL198-0079
BioNova
Reader



Tuttnauer

Innovation · Legacy · Partnership

Tuttnauer USA Co. Ltd.

25 Power Drive, Hauppauge, NY 11788

Tel: +800 624 5836, +631 737 4850, Fax: +631 737 0720

E-mail: info@tuttnauerUSA.com, www.tuttnauerUSA.com



Learn from our experts.

Visit our blog:

<https://tuttnauerusa.com/blog>

