

WI-FI CONFIGURATION

Prior to operating the autoclave for the first time, please connect the Wi-Fi and ensure it remains connected at all times.

The Wi-Fi connection is used for uploading all data information from devices in the field, to the online company's server.



This data is used for:


- Monitoring
- Viewing and downloading device history
- Software updates while validating the integrity of the update (applicable for users with Tuttnauer code)

The data is backed-up on the device main board.



Note: In case of connection failure, the data may be exported manually.

To connect the Wi-Fi:

1. In the **Quick Option** screen, press the **Wifi Configuration** option .
2. To enable the connection, press .
3. The **Wifi Configuration** screen is displayed with the available networks.

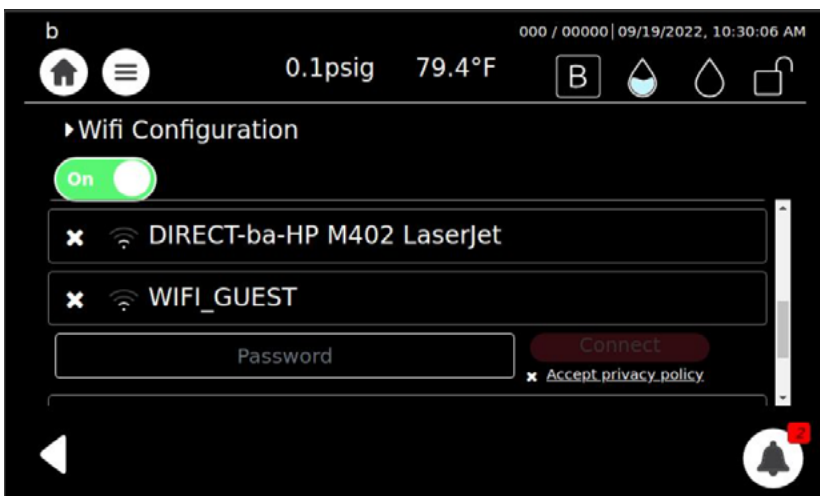

4. Select your network, enter your Password and press the Accept privacy policy.

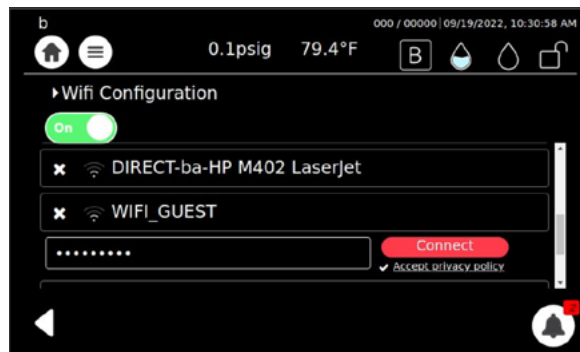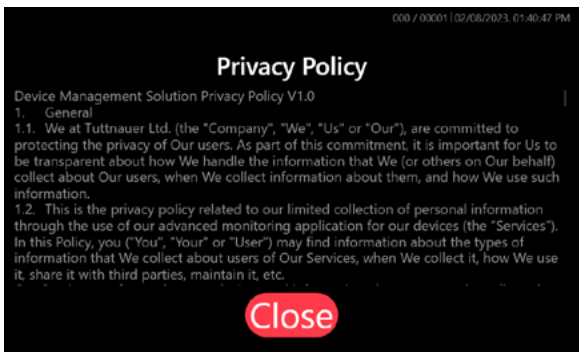

6. The Wi-Fi is connected successfully. Press **Ok**.

USER INTERFACE

