

CHAMBER CLEAN

CLEAN & SHINE
Your T-Edge Chamber

No Mess, No Hassle,
No Measuring

CHAMBER CLEAN tablets – the easiest way to keep your T-Edge clean and free of hard water deposits.

Designed specifically for steam sterilizers, CHAMBER CLEAN effectively cleans and descales the reservoir, chamber and tubing in minutes.

- Simple to use premeasured tablets
- Replaces messy liquid detergents
- Increases sterilization efficiency
- Extends autoclave lifecycle
- Odor free and easy to store
- No material residuals in the chamber




Tuttnauer
Innovation · Legacy · Partnership


CHAMBER CLEAN SIMPLE STEPS FOR PERFECT RESULTS



1. Place one cleaning tablet at the back of the autoclave chamber and close the door



2. Choose the Chamber Clean cycle

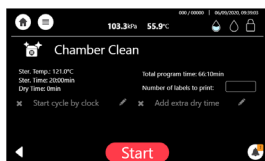

3. Press start to begin the cleaning cycle

4. Post cleaning cycle, sterilization can be applied immediately

- * For additional instructions, please follow directions on the CHAMBER CLEAN box.
- * Use 1 CHAMBER CLEAN tablet in T Edge Autoclaves models. For optimal results it is recommended to run a cleaning cycle using CHAMBER CLEAN every 25 cycles (based on average of 5 cycles a day and 25 cycles a week).

You take care of your patients and leave everything else to us

Renowned global leaders in infection control solutions with more than

90 years'
experience

Products distributed to clinics, hospitals and research institutions in more than

140
countries

International Sales and Marketing

E-mail: info@tuttnauer.com
www.tuttnauer.com

Tuttnauer Europe b.v.
Hoeksteen 11, 4815 PR
PO Box 7191, 4800 GD Breda
The Netherlands
Tel: +31 765 423 510
Fax: +31 765 423 540
E-mail: info@tuttnauer.nl

Tuttnauer USA Co.
25 Power Drive,
Hauppauge, NY 11788
Tel: +800 624 5836, +631 737 4850
Fax: +631 737 0720
E-mail: info@tuttnauerUSA.com

Distributed by:

Tuttnauer
Innovation · Legacy · Partnership

Learn from our Experts
Join our blog tuttnauer.com/blog

